

# Baxter

## SimplFusor

ELASTOMERIC PUMP

# HELP SIMPLIFY INFUSIONS WITH SIMPLFUSOR

A portable medication delivery system designed to be uncomplicated, reliable, and adaptable to therapy and lifestyle needs.

With **SimplFusor** Elastomeric Pumps, you can give your infusion patients the mobility and freedom they need to help maintain independence at home. An elastomeric pump is also well-suited for alternate sites of care including long-term care and outpatient chemotherapy centers.

## HOW SIMPLFUSOR WORKS

The **SimplFusor** Elastomeric Pump has a balloon-like reservoir to be filled with the drug or fluid intended for infusion. It exerts mechanical pressure thereby administering the contents at a predetermined flow rate. The entire unit is sterile. When filled at nominal volume, flow accuracy is within +/-15% of nominal flow rate when delivering normal saline at 88°F (31°C).

The **SimplFusor** Elastomeric Pump is designed to be easy to fill and comes in a wide variety of sizes and flow rates. Each model is color coded by flow rate to facilitate accurate device identification (see chart on back).

## WE'RE WHERE YOU NEED US

Baxter offers a broad range of trusted products, backed by a dedicated sales team, as well as decades of consistent growth and innovation in Home Infusion. We are committed to providing a wide portfolio, seamless ordering, and unparalleled customer support so your Home Infusion practice can support patients' lifestyle needs.



## CHOOSING THE RIGHT SIMPLFUSOR ELASTOMERIC PUMP

The range of models available offers dosing flexibility to administer various infusion therapies. Please refer to the Fill Volume and Delivery Timetables below to help determine which product is best suited for your patient.

	FILL VOLUME AND DELIVERY TIME					
	MODEL NO	PCS/BOX	DURATION	NOMINAL FILL VOL (ML)	RATE (ML/H)	COLOR
SHORT DURATION INFUSION (SD)	481033BAX	48	60m	50	50	Dk Green
	481093BAX	48	120m	100	50	Dk Green
	481043BAX	48	60m	100	100	White
	481013BAX	48	30m	100	200	Turquoise
	481242 BAX	24	120m	200	100	White
	481142BAX	24	60m	200	200	Turquoise
	481152BAX	24	300m	250	50	Dk Green
	481122BAX	24	150m	250	100	White
	481132BAX	24	120m	250	125	Lt Yellow
	481062BAX	24	90m	250	175	Dk Purple
	481052BAX	24	60m	250	250	Dk Grey
	481022BAX	24	30m	250	500	Violet
	481232BAX	24	90m	270	175	Dk Purple
	481102BAX	24	240m	400	100	White
	481072BAX	24	120m	400	200	Turquoise
481082BAX	24	120m	500	250	Dk Grey	

	FILL VOLUME AND DELIVERY TIME					
	MODEL NO	PCS/BOX	DURATION	NOMINAL FILL VOL (ML)	RATE (ML/H)	COLOR
CHEMOTHERAPY INFUSION*	484012 BAX	24	50h	100 ml	2	Yellow
	484032BAX	24	30h	120 ml	4	Orange
	484022BAX	24	135h	270 ml	2	Yellow
	484042BAX	24	54h	270 ml	5	Brown
	484052BAX	24	27h	270 ml	10	Lt Green
	484252BAX	24	50h	300 ml	6	Pink

\*Amber colored outer cover.

	FILL VOLUME AND DELIVERY TIME					
	MODEL NO	PCS/BOX	DURATION	NOMINAL FILL VOL (ML)	RATE (ML/H)	COLOR
LONG DURATION INFUSION (LD)	480052BAX	24	30h	60	2	Yellow
	480012BAX	24	12h	60	5	Brown
	480182BAX	24	200h	100	0.5	Green
	480082BAX	24	50h	100	2	Yellow
	480612BAX	24	20h	100	5	Brown
	480062BAX	24	30h	120	4	Orange
	480032BAX	24	25h	125	5	Brown
	480162BAX	24	135h	270	2	Yellow
	480132BAX	24	68h	270	4	Orange
	480092BAX	24	54h	270	5	Brown
	480042BAX	24	27h	270	10	Lt Green
	480142BAX	24	100h	400	4	Orange
	480112BAX	24	80h	400	5	Brown
	480072BAX	24	40h	400	10	Lt Green
	481172BAX	24	25h	500	20	Dk Violet

MODEL NO	PCS/BOX	DURATION	NOMINAL FILL VOL (ML)	RATE (ML/H)	COLOR
416050BAX/ Softshell Carry Pouch	50	-	-	-	-

Call 888-229-0001 or visit [bax.to/homeinfusion](http://bax.to/homeinfusion) to learn more.

### SINGLE USE ONLY

Elastomeric **SimplFusor** Elastomeric Pump is designed for optimal performance, effectiveness, and safety as a single-use device and not for reuse. Performance, effectiveness, and safety may be compromised if the device is reused. CAUTION: Federal (USA) Law restricts this device to be sold by or on the order of a physician.

### Indications

The **SimplFusor** Elastomeric Pump (Long infusion time article) is intended for continuous infusion of medications for general infusion use including pain management.

- Routes of administration: intravenous and subcutaneous.

The **SimplFusor** Elastomeric Pump (Short infusion time article) is intended for continuous infusion of medications for general infusion use including antibiotic delivery.

- Route of administration: intravenous.

The **SimplFusor** Elastomeric Pump (Chemotherapy article) is intended for continuous infusion of medications for chemotherapy.

- Routes of administration: intravenous and intra-arterial.

### Contraindications

- Infusion of insulin, blood or blood products, TPN, lipids or fat emulsions.
- Infusion of any solutions that are not compatible. Consult the pharmaceutical manufacturer's precautions and guidelines to ensure that the medications used will not interact with the device that may possibly cause damage, leakage or precipitation.
- Intra-articular infusion of local anesthesia.
- Infusion of anesthetics in neonates, infants and children below 5 years of age.

**Rx only.** For safe and proper use of the products mentioned herein, please refer to the appropriate Operator's Manual or Instructions for Use.

### Baxter.com